

Appendix Remote control pairing operation instructions

There are 2 remote controls distributed with the vehicle, among which the remote control placed in the packaging of the net pocket is the standby remote control. To ensure safe use, this folding controller can only pair one remote control, and the backup remote control only exists as a backup. If you confirm that you need to use the spare remote control, please follow the following instructions.



To ensure safe operation and use, the folding controller designed by us can only pair one remote control. So only one remote control can be used with the folding controller from beginning to end.

The pairing process is as follow:

1. Make sure the battery is powered up when pairing (you can confirm whether it is powered up by pressing the power switch of the operating handle)
2. Separate the harness connector of the remote control mounted on the vehicle frame from the control box



3. Re-insert the unplugged connector into the power supply position of the control box
4. Press any button of the remote control within 5 seconds (S) after inserting the power harness and hold it for 5 seconds before releasing the button.

Power folding button A
Power folding button B
Power un-folding button C
Power un-folding button D



5. Press the remote control to confirm whether the function of the remote control is working.
6. If the remote control does not respond, please refer to the above steps 1-4 to match again