

ROLLATOR USER MANUAL



Dear Customer

Thank you for your purchase of our rollator.
Please find the following instructions for use and product specifications.
We wish you all the best with your new rollator.

Contents

Safety Guidelines	1
Product Specifications	2
Summary	3
Assembly and User Instructions	4
Maintenance and Other	10

Safety Guidelines

The symbols below are used throughout this owner’s manual to identify warnings and important information. It is very important for you to read them and understand them completeley.



WARNING! Indicates a hazardous situation that could result in serious injury or death if it is not avoided.



CAUTION! Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.



IMPORTANT! Indicates a hazardous situation that could result in damage to property or rollator if not avoided.

Product Specifications

The rollator has been designed to assist the user to walk, rest and carry small items of shopping for both indoor and outdoor use on flat surfaces by a single individual with limited mobility with a maximum weight as in the below table.

It is important that the user reads and understands the safety advice given within this manual before attempting to operate the product to avoid possible injury to the user or bystanders and avoid damage to the wheeled walker or surrounding property.

User Weight (Max)	21 st. (136 kg)
Total Weight	5.65 kg
Handle Height	810-910 mm
Seat Width	460 mm
Seat Height	560 mm
Chassis Type	Lightweight Aluminium
Overall Width Folded	250 mm
Overall Width Open	625 mm
Overall Length	730 mm
Wheel	EVA Soft Roll
Wheel Size	200 mm
Stick Holder	Yes
Storage Bag	Yes
Padded Backrest	Yes
Locking Band	Yes

Summary

The Rollator has been manufactured from high quality materials, aluminum frame, PP, nylon seat, Polyester bag and locking brake. The handles are adjustable in height and the wheeled walker folds easily for transport or storage.

The Rollator has been designed to assist the user to walk, rest and carry small items of shopping for both indoor and outdoor use on flat surfaces by a single individual with limited mobility with a maximum weight as shown opposite.



1 Hand Grip

2 Back Rest

3 Brake Lever

4 Pull Strap for Folding

5 Seat

6 Handle Tightening Knob

7 Rollator Bag

8 Rear Wheel

9 Stick Holder

10 Front Wheels

Assembly and User Instructions




WARNING!

Ensure that all bolts are securely fastened before using this rollator!

Brakes must be in the parked /locked position before sitting on the rollator!

1. Remove all packaging and set out parts

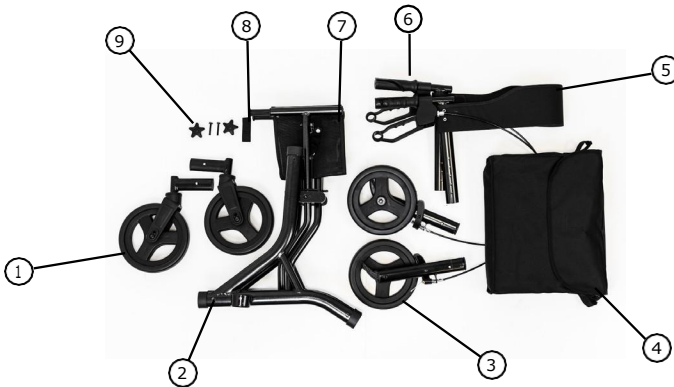

1 Front Wheels

2 Frame

3 Rear Wheels

4 Rollator Bag

5 Back Rest Strap

6 Handles

7 Seat

8 Band

9 Tightening Knob

2. Insert front and rear wheels



Install front and rear wheel sets as shown. Put the wheel set tube into the frame tube and make sure the spring ball has popped out. Do the same for all four wheels.

3. Opening rollator



To open the rollator, push down on either side of the cross bars as shown. You should feel both sides click into position.



CAUTION!

Be careful not to trap your fingers when pushing down the seat.

4. Inserting the handles



Insert the handles as shown and push down to the required height. Use the tightening knobs provided to secure in place at the desired height.

5. Folding the rollator



To fold the rollator pull up the strap in the middle of the seat as shown.



Note: As shown, a band is supplied to hold the rollator together in folded position

6. Rollator Bag



Attach the bag to the rollator by placing the bag hoops over the two bag hangers positioned at the front of the seat as shown. Place the two Velcro bag straps around the rollator frame to secure the lower section of the bag.



CAUTION!
Never overload the bag's 5kg MAX load.

7. Back Strap



Before using the rollator check that the back strap frame bolts are securely tightened and that the velcro straps are secure.

8. Stick Holder



Attached to the rollator is a stick holder. Place your stick into the stick holder as shown and secure stick by using the Velcro strap attached to the back rest.

9. Handle Height Adjustment



To adjust the handle height up or down, undo the tightening knob and adjust the height. Tighten the knob back up making sure the handles are secure and level.

10. Braking



For braking while moving, pull both brake levers upwards as shown and always brake slowly and steadily.

11. Locking the Brakes (Parking Position)



Push both brake levers down to apply and lock the brakes while the rollator is stationary.



WARNING!
Test the brakes regularly

12. Sitting on the Rollator

Before sitting down on the rollator ensure the brakes are in the parking position and make sure the rollator is on an even surface and will not slide or topple over. Sit up straight and never lean backwards.

Do not support your weight on the handle when attempting to sit. Use the handle to stabilize your body as you carefully turn around. Keep your hands on the handles and slowly lower your weight onto the seat.



WARNING!
Never move rollator while seated.

Always apply parking brakes before sitting.

Maintenance and Other



IMPORTANT! Do not use strong corrosive cleaners to clean or polish surface. This could damage the rollator.



IMPORTANT! Store in dry location.



CAUTION! Be careful when folding and unfolding the rollator to avoid injury by the mechanism.



WARNING! Always apply brakes when in stationary position or when seated on rollator.



WARNING! Do not hang bags or other items from handles - this will add stress to the rollator and could also make it unstable.

